ISOGL203

# User Information

## POWDERED LATEX EXAMINATION GLOVES NON STERILE

Available size : S/7, M/8, L/9, XL/10

Manufacturer Address :

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### 1 - EU Type-Examination

**a** - This product is classified as Category I Personal Protective Equipment (PPE) according to PPE Regulation (EU) 2016/425 and has been shown to comply with this Regulation through the Harmonised European Standards EN ISO 21420:2020

**b** - This product is classified as a Class I medical device, in accordance with EU Regulation 2017/745, Annex VIII, Chapter III, Rule 1.

**c** - The EU Declaration of Conformity is accessible at [www.mediprotec.com](http://www.mediprotec.com)

### 2 - EN ISO 21420:2020

Clause 5.1 - Length and fit : PASS

Clause 5.2 - Dexterity : Level 5

Glove size	7 - S	8 - M	9 - L	10 - XL
Minimum length / mm	230	240	250	260

Performance level	1	2	3	4	5
Diameter of dexterity pin / mm	11.0	9.5	8.0	6.5	5.0

### 3 - Product Instruction for Use

**a - Usage** - For Single Use only.

**b - Sizing** - Select the right size glove for your hand

**c - Donning** - Prior to donning, perform hand hygiene using hand rub or antimicrobial solution with water. Hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to get a good fit. Don the other glove by the same procedure.

**d - Inspection** - Punctures or tears may occur after donning. Inspect each glove after donning, and immediately discontinue use if found damaged.

**e - During Use** - Gloves polluted by blood or other organic fluids must be immediately removed as medical waste. In case of pricking or tearing, user should immediately change gloves. In case of feeling looseness on fingers, user should change gloves.

**f - Doffing** - Hold glove bead and pull toward the fingers until glove comes off.

**g - Disposal** - Properly disposal of all used gloves. Follow your Institution's policies for disposal.

### WARNING

- This glove contains natural rubber latex which may cause allergic reactions, including anaphylactic responses. If allergic reactions occur, seek for medical advice immediately.

- For minimal risks only

### 4 - Handling and Storage

Store in a cool and dry place. Opened boxes should be kept away from fluorescent and sunlight. Gloves are packed in boxes which is suitable for transport. Keep the gloves in the box when not in use.

### 5 - Shelf life

The shelf life of product is 5 years from date of manufacture.

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